

NEW! FDA-APPROVED

HYLAFORM®

(HYLAN B GEL)



CORRECT WRINKLES BY
RESTORING SKIN VOLUME

 **INAMED**
AESTHETICS


HYLAFORM® COSMODERM® COSMOPLAST® ZYDERM® ZYPLAST®

A KEY COMPONENT OF BEAUTIFUL SKIN

Healthy skin has an abundance of two important substances, collagen and hyaluronic acid, that keep your skin looking smooth and supple. Collagen is a protein that provides your skin with structural support. Hyaluronic acid is a natural sugar found in all living cells that attracts and binds water, hydrating the skin and giving it volume.

With time, sunlight and other factors, collagen and hyaluronic acid in your skin diminish, the skin loses structure and volume, and unwelcome lines and wrinkles appear.

Now you can temporarily replace your skin's lost hyaluronic acid using Hylaform® (hylan B gel). Hylaform is a dermal (skin) filler made from a highly purified form of hyaluronic acid that mimics the effects of natural hyaluronic acid. When injected just below the surface of the skin, Hylaform adds volume to the skin, smoothing unwanted lines and wrinkles.

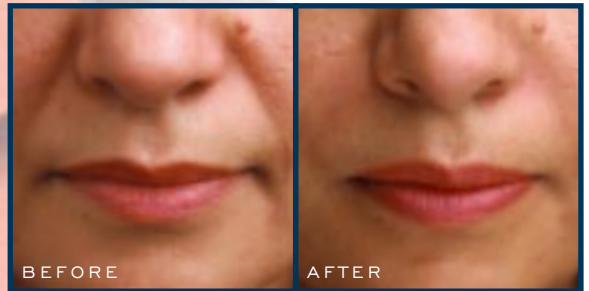

Smile lines




Marionette lines

IMMEDIATE RESULTS WITH FAST RECOVERY

Hylaform® provides immediate, natural-looking results. Treatment and recovery usually take as little as one hour and no pre-treatment skin test is required.

- Immediate treatment—no pre-treatment skin test required
- Immediate results
- Proven safety—used in international markets for nearly a decade
- Common side effects are mild
- Has been used in combination* with collagen-based dermal fillers

Hylaform can be used to fill in smile lines between the nose and mouth, small lines above the lip, frown lines between the eyes, and to minimize creases that turn down the corners of the mouth.

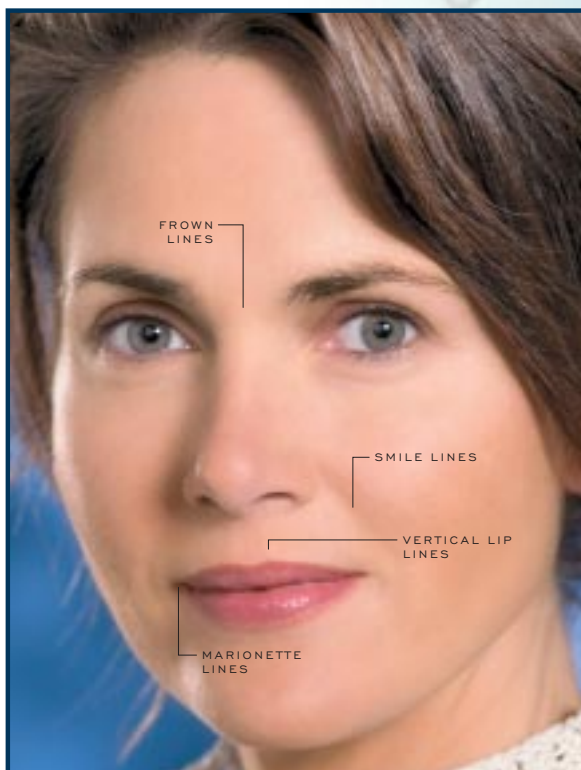

ASK YOUR PHYSICIAN

To find out more about Hylaform® and its uses alone or in combination*, please ask your physician. He or she can work with you to develop a treatment program to meet your needs. For full prescribing information, please visit www.inamed.com.

Hylaform is brought to you by INAMED Aesthetics, the only company in the world with more than 20 years of experience providing safe and effective dermal filler treatments.

HYLAFORM® (HYLAN B GEL)

Hylaform®—Important Treatment Considerations for Consumers

In the U.S., Hylaform gel is indicated for injection into the mid to deep dermis for correction of moderate to severe facial wrinkles and folds (such as nasolabial folds), and is generally well tolerated. The most commonly reported side effects are temporary injection site redness, bruising, swelling, pain and itching. Other side effects, such as lumps/bumps and tissue hardening have been reported rarely. Exposure of the treated area to excessive sun, UV lamp, and extreme cold weather should be minimized until any initial swelling and redness has resolved. If procedures such as laser treatment or a chemical peel are used before or after Hylaform gel treatment there is a possible risk of an inflammatory reaction at the implant site. This also applies if Hylaform gel is administered before the skin has healed completely after such a procedure. Hylaform gel should not be used in patients with known susceptibility to keloid formation, hypertrophic scarring or pigmentation disorders.

Patients using substances that can prolong bleeding may experience increased bruising or bleeding at injection sites. As with all transcutaneous procedures, Hylaform gel use carries a risk of infection and standard precautions used for injectable materials should be followed. Hylaform gel must not be injected into blood vessels; doing so may cause occlusion of the vessels and could cause infarction or embolization. Hylaform gel should be used with caution in patients on immunosuppressive therapy.

Hylaform gel is not intended for use in patients with a history of known allergy to avian (bird) proteins. The use of Hylaform gel in areas of the skin with any infection or inflammatory process (e.g., cysts, pimples, rashes or hives), should be postponed until the conditions have been resolved. The safety of Hylaform gel in persons younger than 18 years of age, or in pregnant or breastfeeding women has not been tested. The safety and efficacy of Hylaform gel for use in lip augmentation and regions other than nasolabial folds has not been established. The safety of injecting greater than 20 mL of Hylaform gel per 130 lbs body mass per year has not been established. Long-term safety and effectiveness of Hylaform gel beyond one year has not been investigated in clinical trials.

M933-01 4/04





RESTORE SKIN VOLUME

- NEW Hylaform® replenishes the skin's lost hyaluronic acid, restoring skin volume
- Immediate treatment—no pre-treatment skin test required
- Immediate results
- Proven safety—used in international markets for nearly a decade
- Common side effects are mild
- Has been used in combination* with collagen-based dermal fillers and other products

To find out how INAMED Aesthetics' products can temporarily reverse the visible effects of time, ask your physician, or contact:



5540 EKWILL STREET
SANTA BARBARA, CA 93111, USA
TEL: 800.624.4261
WWW.INAMED.COM

1235 BAY STREET, SUITE 900
TORONTO, ON M5R 3K4, CANADA
TEL: 877.485.7535

*The safety and effectiveness of combination use has not been clinically evaluated.

© 2004 INAMED Corporation. INAMED, the INAMED logo, CosmoDerm, CosmoPlast, Zyderm and Zyplast are trademarks or registered trademarks of INAMED Corporation. Hylaform is a registered trademark of Genzyme Corporation.
M935-01 6/04